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Lancing Device Instructions for use:

Caution: Never share a lancet or lancing device with another person. Always use a new, sterile lancet. Lancets are for single use only. Not for multi-patient or clinical use!

1. Pry the depth adjustment cap from the shank.

2. Insert a new lancet into holder. Push until the lancet stops completely.

3. Twist off the lancet's safety cap. Do not bend the needle, and save the safety cap for lancet disposal (see below).

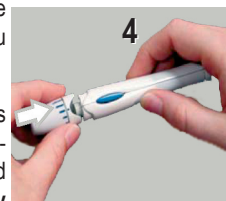
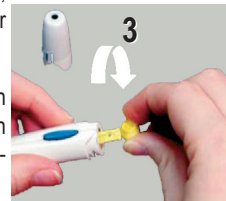
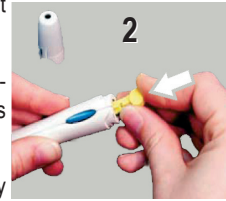
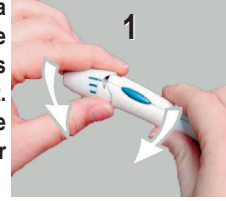
4. Carefully replace the depth adjustment cap as a shown and avoid contact with the exposed needle.

5. Pull the charger firmly at the end to reset the device. You may hear a „click“.

Be aware of all hygienic rules and sampling recommendations, given in the attached documents! **Wear gloves by treating another person!**

6. Hold the lancing device firmly against the area (earlobe or fingertip) to be lanced. Push the release button, and the needle will puncture the skin.

After the measurement open the lancing device carefully and stick the lancet onto its safety cap (see pt. 3) before remove: Used lancets and other disposables must be disposed safely. Items contacted by blood must be disinfected.

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General hints for sampling (1):

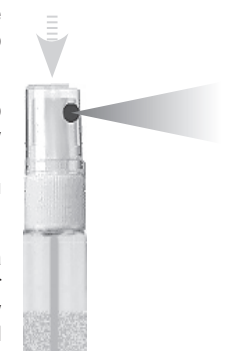
Puncturing the skin and general handling of blood includes always the risk of infections: Before sampling it's strictly recommended to disinfect the skin on the sample area and also the tip of the lancing device carefully.

The main reason for wrong lactate readings is the contamination of the blood by sweat: Human skin is covered with sweat, which contains a great deal of lactate. Due to the density of sweat glands, especially in the fingertips, sweat is continuously secreted and accumulated at the surface. Alcohol doesn't dissolve the lactate of sweat. In addition, do not use saliva for cleaning, because it also contains lactate.

Ensure that the area (fingertip or earlobe) is cleaned carefully with water before sampling. E.g., you can wash your hands or you take a water bottle for cleaning.


The attached spray mister is a handy cleaning tool for outdoor application: Please keep the spray mister always clean, and refill with clean water only, without other liquids or additives! Also replace the protection cap after use.

To avoid other contaminations or dilution of the blood drop after sampling, dry the puncture area after cleaning/disinfection thoroughly. Cleaning swabs or a clean tissue can be used both.

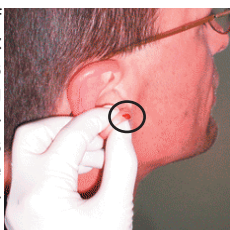Low temperatures or a poor flow of blood may cause an insufficient small drop of blood that forms after sampling. For improving sampling at the earlobe we recommend stimulating cremes, as available e.g. in pharmacies. The treated sample area must be cleaned very carefully afterwards!

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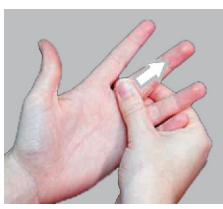
General hints for sampling (2):


For sampling at the fingertip it's helpful to wash the hands with warm water or to hold the hand down. Also a gentle massage from the base to the tip of the finger will obtain the required blood volume.

Press slightly the finger, but do not squeeze the puncture area to increase the flow of blood: This may contaminate the sample with sweat!

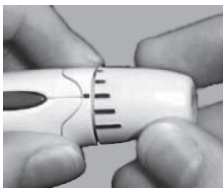For a sufficient blood sample and reducing the pain, lance on the sides of the fingertips in the areas shown. Don't use the 2nd finger because its continual contact with other items.

For the different thickness of skin the depth of puncture is adjustable in 5 settings. Twist the depth adjustment at the front of the lancing device: As longer the marker line, the deeper the puncture will be.

Make sure to wear protection gloves if another person is involved in any application!

Hurry up with the sample uptake after puncturing - the delay time between drop generation and measurement should be very short: Within a few seconds, physical and chemical parameters of the generated blood drop and sweat will be produced permanently which increases again the lactate concentration on the skin.

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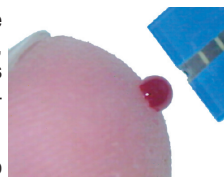
General hints for sampling (3):

The attached lancing device can be used with standard blood lancets as available in drugstores and pharmacies. For medical application it's usual to operate with single-use lancets, for lancing manually. Safe and painless puncturing is also practicable by single-use lancing devices („safety lancets“).


Sampling with comfort lancing devices causes only a little flow of blood. Manually used lancets can produce a continual flow of blood: They should be used only by qualified staff, regarding strictly all rules for hygienic and safe application.

The bigger the blood drop is formed, the lower is the part of potential contamination. If the blood sample doesn't form a drop, clean the sample area again and repeat it. It's recommended to wipe away the first drop that forms by a clean tissue: The next drop should be used for the measurement. In this way, the risk of influences by sweat or alcohol rests will be reduced.

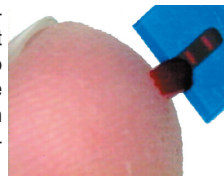Lower the tip of the inserted test strip from top down to the fresh drop of blood. Avoid smearing the blood sample by the test strip: The drop may collapse, and the sample will not be uptaken properly. Do not touch the skin by the tip of the test strip, which could cause to wrong results by sweat influence.

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